


TL-2100S Safety Auto-Reset Fall Monitor

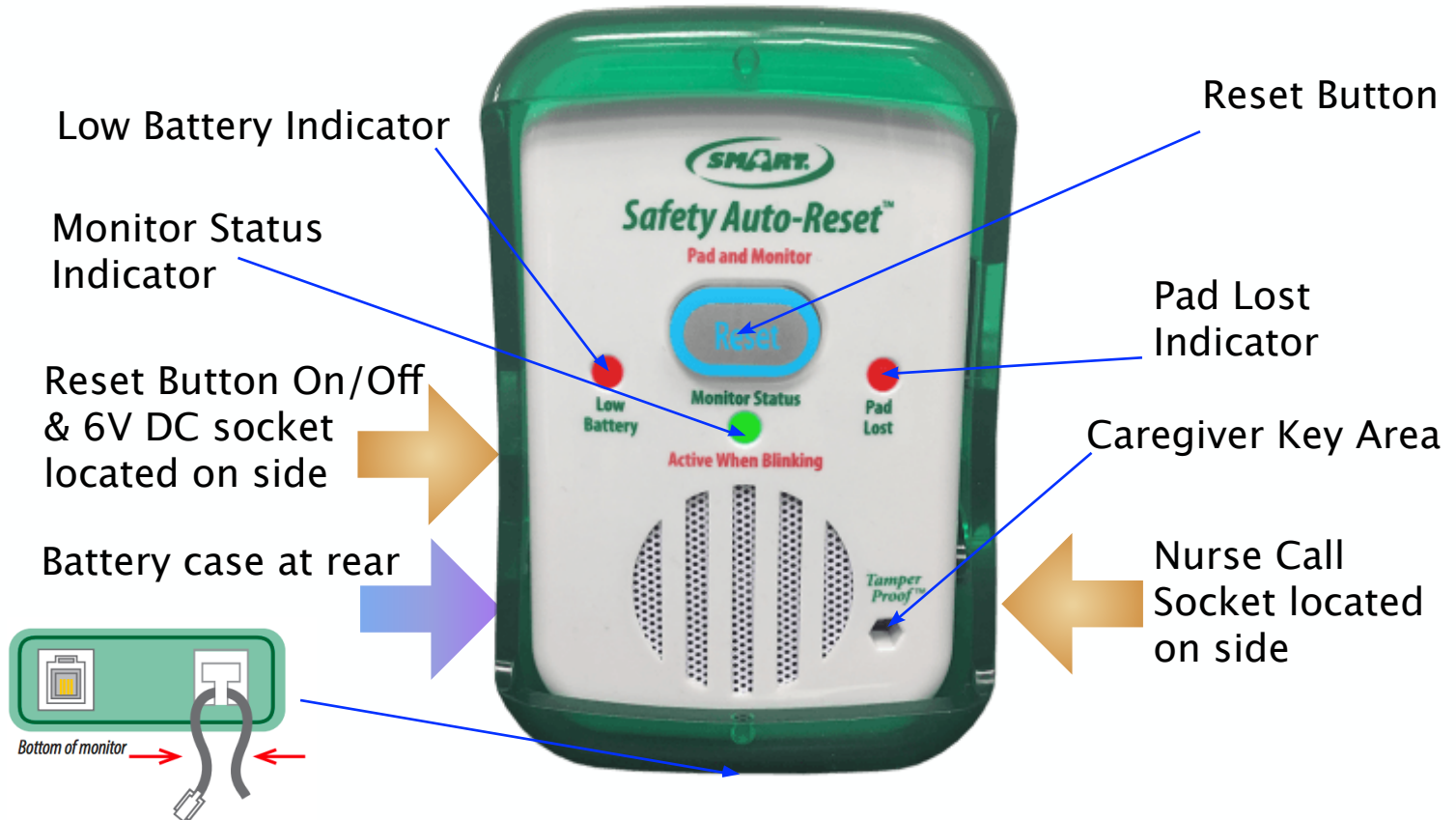

Programming/ Setup

- ◆ Remove protective boot, undo screw on battery cover place 3x AA batteries in rear & adjust volume & tone and delay. Replace screw. Replace boot.
- ◆ **Connect the corded pressure pad, seat belt or floor mat (socket located at bottom of monitor –run cable through strain relief recess)**
- ◆ Testing – Put pressure on the sensor pad to activate. Monitor will beep twice. The status light will blink when pressure is on the pad. When pressure is removed, alert tone will sound. Press reset button on front to silence / reset

Notes

- ◆ For use with one corded bed pad, chair pad, floor mat or seat belt
- ◆ Volume dial, tone and delay settings inside battery case at rear of monitor
- ◆ Reset switch on side of monitor – "Reset Switch On" means the reset button on front of monitor is active. "Reset Switch Off" means Caregiver Key point at front of monitor is active, and reset button on front is inactive.