


433-CMU-40 Economy Central Monitor

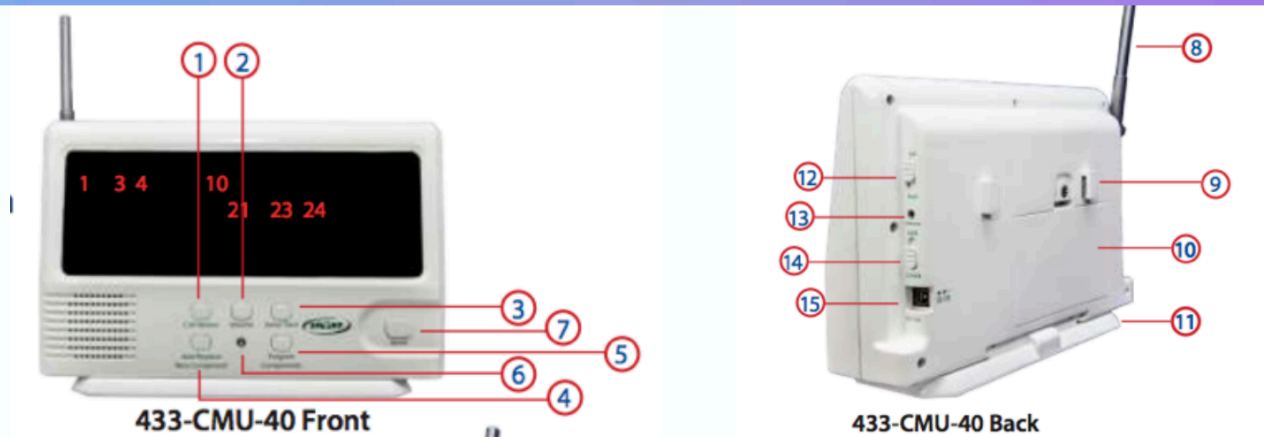

- 1) Call Review Button. 2) Volume Button. 3) Voice/Tone Button. 4) Add/Replace New Component Button. 5) Programme Components Button. 6) Programming Light. 7) Reset Button. 8) Antenna. 9) Hook. 10) Battery Compartment. 11) Base. 12) Call/Alarm Switch. 13) Silencer Port. 14) Lock/Unlock Switch. 15) AC Adaptor Input

Programming / Setup

1. Insert 4x "C" size batteries in rear battery compartment (10) (remove screw on battery cover to access) or plug in AC adaptor.
2. For new bed or chair pads, apply 30 seconds of pressure to initialise the pads
3. Put the 433-CMU into the unlock position (14)
4. Select ALARM or CALL (12) on the side of the CMU. ALARM is a siren sound. CALL is the tone selected using the VOICE/TONE button (Recommended)
5. Select the desired tone by pressing the VOICE/TONE button (3)
6. Adjust VOLUME button (2) to desired volume.
7. Place components in order you want to programme them (pads should be on a flat surface and all items within reach)
8. Hold down the PROGRAMME COMPONENTS button (5). Wait until you hear two dings and a melodic tone, then release button.
9. Activate first component:
Bed or Chair Sensor Pad – apply pressure and release after 2-3 seconds
Nurse Call Button – push button and then release (red light on call button will illuminate)
Motion Sensor – turn switch on sensor to on and wave hand in front of sensor (this could take up to 15 seconds) then turn off when the CMU calls the number or sounds the selected tone earlier
Pager with Reset Button – press and release reset button on the pager
Floor Mat – turn on the floor mat transmitter and step on floor mat until CMU alerts
10. The CMU LCD display will numerically show and speaker will audibly announce the component alert number. When programming pager with reset button, the CMU will ding but not show a number

11. Wait for the CMU to sound a melodic confirmation tone before going to the next step

12. To programme additional component(s) repeat steps 7-10 for each new component

◆ IF CMU TIMES OUT AFTER PROGRAMMING FIRST COMPONENT(S) REFER TO **ADDING ADDITIONAL COMPONENTS** SECTION IN BLUE BELOW

13. After you have activated the last component, wait for CMU to sound its melodic confirmation tone and then move the switch (14) to the LOCK position and press and release the RESET button (7) on the side of the CMU

14. Test the system. Activate each component in turn, wait for alert then press RESET button. System is now ready for use!

ADDING ADDITIONAL COMPONENTS

1. Ensure no components are being activated when adding additional components to the CMU
2. Place new components in the order you would like to programme them. Bed and chair pads will need to be on a hard surface and switched on
3. Put the CMU into the UNLOCK position and then press the ADD/REPLACE NEW COMPONENT button (4) for two dings and a melodic confirmation tone then release
4. Press the PROGRAMME COMPONENTS button (5) once and release. Wait for melodic tone
5. Flashing number on screen is the last number that a component is currently programmed to.
6. Activate components as "Activate First Component" above
7. Wait for melodic confirmation tone and then put CMU into LOCK position
8. Press and release RESET button (7) on the CMU. Test components.

FOR FURTHER INFORMATION REFER TO THE SMART CAREGIVER 433-CMU-40/60 INSTRUCTION BOOK SUPPLIED WITH THE WIRLESS CMU (CENTRAL MONITORING UNIT), OR CALL THE FRIENDLY AND KNOWLEDGEABLE STAFF AT HEALTHSAVER ON 1300 767888

WWW.HEALTHSAVERALARMS.COM.AU

