


433-EC Economy Cordless Fall Monitor

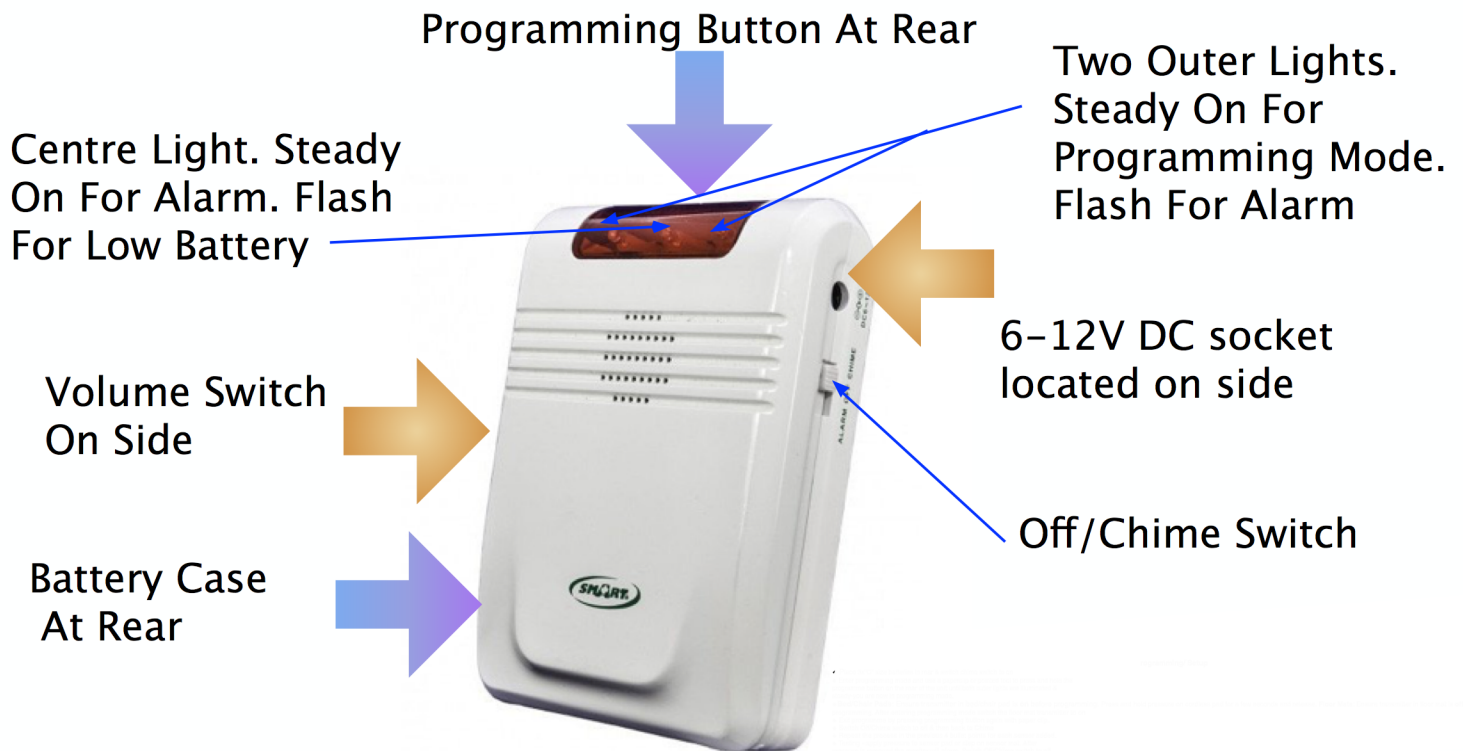

Programming/ Setup

- ◆ Place 3x"C" size batteries in rear & switch chime switch to on
- ◆ Enter programming mode and use a paperclip or pointed tool to press and hold the programme button on the rear of the unit until both outer lights are illuminated & steady-you are now in programming mode.
- ◆ **Bed/Chair Pads:** Ensure transmitter in bed/chair pad is **on** before programming. Press and hold pressure on cordless pad for a few seconds and release. **Floor Mats:** Ensure transmitter in floor mat is **off** before programming. After entering programming mode switch the floor mat transmitter to on
- ◆ Exit programme by pressing programming button again with paper clip.
- ◆ Switch Off/Chime switch to off & then back to Chime
- ◆ Repeat the process in the previous 4 bullet points for each sensor added.
- ◆ Testing - apply pressure to sensor pad or step on sensor mat. After pressure is removed the monitor will alarm. Switch Off/Chime switch to off and back to chime to silence and reset. (System will also re-set and silence alarm if pressure is reapplied to the sensor pad)

Notes

- ◆ For use with up to 6 cordless bed pads, chair pads, floor mats, motion sensors or nurse call/reset buttons.
- ◆ 100m maximum distance between monitor & sensors
- ◆ To clear all programming hold the programme button on back of monitor down until the LED lights on the front are blinking. Release. Programming is now cleared.